

ROSACEA

Roadmap for Results

A simple guide to help you flush out the red for a flawless finish using the best DermapenWorld has to offer.



Dermapenworld TREATMENT JOURNEY

CHARACTERISTICS

Visible red capillaries
Raised skin patches
Persistent Blushing
Swollen bumps

SKIN PREPARATION

1

Cleanse



TRI-PHASE
CLEANSER™

2

Sanitise



CLINIPREP™

MESO-GLIDE™ APPLICATION



MG-HA35+™ for extreme hydration
or the potent blend of MG-R.A.S.™
for skin-enhancing effects.



Consistent pinpoint blood
(papillary/reticular dermis)

VISUAL ENDPOINT OF MICRONEEDLING

Select top speed

MICRONEEDLING DEPTH & SPEED (GUIDE ONLY)

Select appropriate needle depth
per treatment area eg 0.3mm for
peri-orbital, up to 1.5mm for cheeks.

TREATMENT UPGRADE

MG-EXO-SKIN™
High-performance
Meso-Glide™

TREATMENT FREQUENCY

4-6
treatments

or until desired results
are achieved

6-8
week intervals

NON-NEGOTIABLES FOR PATIENTS

POST-TREATMENT CARE

Reduce redness & downtime



HYLA ACTIVE™
3D SCULPTURED
MASK



VITAMIN RICH
REPAIR™



COVER
RECOVER™

AT-HOME AFTER CARE

Soothe, nourish
& protect skin



TRI-PHASE
CLEANSER™

Cleanser specifically
formulated to improve
skin conditions,
such as rosacea.



ANTIOXIDANT
COCKTAIL™

Corrective serum
combats uneven
skin tone with
melanosome transfer
inhibiting qualities.



COVER RECOVER™

A daily
broad-spectrum
SPF30/PA+++
camouflage and
skin protectant.

TREATMENT UPGRADE

Dermapen HOME™
At-home microneedling.
Patients can maintain
results between
appointments.

Practice Protocols

For ultimate results always follow
DermapenWorld Protocols.

Access protocols for rosacea.

www.dermapenworld.com/my-account/atp-zone-marketing

WORLD'S #1
MICRONEEDLING
COMPANY

Dermapenworld™